



January 2023

Dear Healthcare Provider,

Baxter is committed to supporting your patients every step of the way on their PD journey. In the spirit of continuous improvement, the Amia system software will soon be updated to version 2.10.10.

This letter will explain the changes incorporated with the Amia APD system software and what to expect to complete the update for your patients.

Included Products

Product Code	Description	Serial Numbers	Distribution Dates
5C9320	Amia APD system	All	Starting in February and over the coming months

Why is the software changing?

As a pioneer in remote patient management, Baxter's data communication system powering Sharesource has been hard at work these past years, transferring data with safety and security in mind. A new IT infrastructure is coming to maintain our high standards for data security and safety. To prepare for this new IT infrastructure, your patients' cyclers require new software. Rest assured, the Amia cycler experience and your Sharesource experience are unchanged by the update. Please note that this important update is required for cyclers to connect to Sharesource moving forward.

When will the software update occur?

The new version, 2.10.10, will be arriving via remote deployment to your patients starting in February and will continue over the coming months.¹ We will be informing patients of the upcoming software update and providing contact details for technical support and troubleshooting.

What changes have been made to the software?

The new software updates the Sharesource communication system to improve data management and security. Both communication systems will remain active through 2023. Devices that do not receive the update will be unable to connect to Sharesource after January 2024.

BELOW IS AN OVERVIEW OF HOW TO UPDATE YOUR CLINIC CYCLER(S):

Keep your Clinic Cyclers Connected:


Cyclers must be connected to Sharesource to receive the update. If your connection to Sharesource has been interrupted, you may not receive the software deployment automatically.

What do I need to do to receive this update?

Your new Amia System software will download as soon as it is available. You may defer the installation for one treatment only.



Note: Your software update has started when you see this notification on your Amia touchscreen

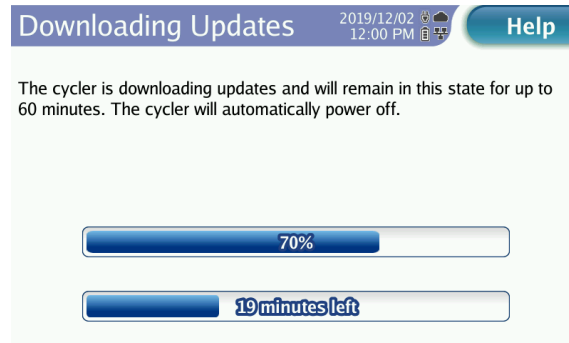

For further information, please refer to the detailed instructions in the Patient at Home Guide (Section 6.3 Cyclor Software Updates Using the Sharesource Connectivity Platform).

If you have any questions, Baxter Technical Support is available by phone at 1-800-553-6898.

Sincerely,

Priscilla Cruz
PD Products
Baxter Healthcare

The Amia Automated PD System is intended for automatic control of dialysate solution exchanges in the treatment of adult renal failure patients undergoing peritoneal dialysis. All therapies using the Amia Automated PD System must be prescribed and performed under the responsibility of a physician who is familiar and well-informed about peritoneal dialysis.

The Sharesource portal is intended for use by healthcare professionals to remotely communicate new or modified treatment parameters with compatible dialysis instruments and transfer completed treatment data to a central database to aid in the review, analysis, and evaluation of patients' historical treatment results. This system is not intended to be a substitute for good clinical management practices, nor does its operation create decisions or treatment pathways.

Rx Only. For safe and proper use of the devices mentioned on this website, refer to the Instructions for Use or appropriate Operator's Manuals.

¹The new software is compliant with the requirements from the latest Industry Standards for Peritoneal Dialysis equipment IEC 60601 2-39 (3rd Edition) and the specific Industry Standard for Alarms IEC 60601-1-8 (2.1 Edition).