

January 2023

Dear Valued Baxter Patient,

Baxter Healthcare Corporation is enhancing the Amia Automated PD System with the next software version. Starting in February and continuing over the coming months, Baxter will begin sending remote updates to Amia Systems through Sharesource to your cyclers.

What changes have been made to the software?

This software update is designed to maintain communication continuity between the Amia cycler and the Sharesource remote patient management portal and help enhance Amia System data security. Devices that do not receive the update will be unable to connect to Sharesource after January of 2024.


Cyclers Requiring a Software Update:

Product Code	Description	Serial Numbers	Distribution Dates
5C9320	Amia APD system	All	Starting in February and over the coming months

WHAT DO I NEED TO DO TO RECEIVE THIS UPDATE?

When the Amia software update is available, you will receive the notification below on your device. Your cycler must be connected to Sharesource to receive the update. If your connection to Sharesource has been interrupted, you may not receive the software deployment automatically. This update is critical, so please note that you may defer the installation for one treatment only.

Note: Your software update has started when you see this notification on your Amia touchscreen

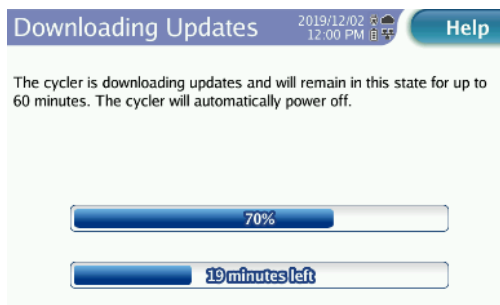

What if I have recently received a replacement device or am just starting treatment? Do I still need a software update?

If you have recently begun treatment using the Amia System or have recently received a device exchange, it is possible that your device is already equipped with the new software. Please refer to the HELP menu on the Amia touchscreen or section 10.7 of Amia Patient Guide to check your system’s software version. If your Amia System has software version 2.10.10 or greater, no further action is required.

I noticed my program needs to be confirmed. What should I do?

Some Amia software updates require your program settings to be confirmed. Verify the settings are correct and follow the on-screen steps to accept the program.



In most cases, you will be able to update your system without additional assistance. You may need to re-enter your unique activation code that you used when you initially activated your system. If you do not remember the code, contact your clinic.

If you have any questions, contact Baxter Technical Services Support at 1-800-553-6898.

Sincerely,

A handwritten signature in black ink that reads "Priscilla Cruz".

Priscilla Cruz
PD Products
Baxter Healthcare

The Amia Automated PD System is intended for automatic control of dialysate solution exchanges in the treatment of adult renal failure patients undergoing peritoneal dialysis. All therapies using the Amia Automated PD System must be prescribed and performed under the responsibility of a physician who is familiar and well-informed about peritoneal dialysis.

The Sharesource portal is intended for use by healthcare professionals to remotely communicate new or modified treatment parameters with compatible dialysis instruments and transfer completed treatment data to a central database to aid in the review, analysis, and evaluation of patients' historical treatment results. This system is not intended to be a substitute for good clinical management practices, nor does its operation create decisions or treatment pathways.

Rx Only. For safe and proper use of the devices mentioned on this website, refer to the Instructions for Use or appropriate Operator's Manuals.