

Baxter

CleanCart

CLEANING AGENT

BAXTER KIDNEY CARE

PD | HD | HDx | EDUCATION | SUPPORT

AN EFFORTLESS WAY TO TRANSFORM DIALYSIS MACHINE MAINTENANCE

CleanCart cartridges make dialysis machine cleaning, decalcification, and disinfection simple, safe and efficient.

Compared to conventional liquid disinfectants, CleanCart cartridges may result in:



LESS COST

Roughly 40% annual savings per machine and minimized shipping costs¹



LESS TIME

Ability to auto-schedule, and no residual testing required^{2,3}



LESS RISK

Minimize risk of human error, material damage and spills³



LESS SPACE

Up to 69% less space and 85% less weight for simpler storage¹



LESS ENVIRONMENTAL IMPACT

Recyclable and more environmentally friendly^{4,5}



CLEANCART CARTRIDGE SPECIFICATIONS

CLEANCART A

The **CleanCart A** cartridge contains 13 grams of anhydrous sodium carbonate powder. The **CleanCart A** cartridge is intended for preparation of a sodium carbonate solution used for removing organic deposits, fats and proteins from the dialysis machine's fluid circuit. The **CleanCart A** cartridge must be used in combination with the heat disinfection program of the corresponding dialysis machine.⁴



CLEANCART C

The **CleanCart C** cartridge contains 32 grams of citric acid anhydrous powder. The **CleanCart C** cartridge is intended for preparation of a citric acid solution used for removing calcium and magnesium precipitation from the dialysis machine's fluid circuit. The **CleanCart C** cartridge must be used in combination with the heat disinfection program of the corresponding dialysis machine.⁵



| TECHNICAL DATA | CLEANCART A | CLEANCART C |
|--|--|---|
| Action in combination with the heat disinfection program | Cleaning of the fluid path from organic deposits, fats, and proteins | Decalcification of the fluid path |
| Cartridge content | 13 grams | 32 grams |
| Total cartridge weight | Approx. 32 grams | Approx. 50 grams |
| Cartridge and label material | Recyclable polypropylene (PP) | Recyclable polypropylene (PP) |
| Recommended usage | Refer to operator's manual for corresponding dialysis machine | Refer to operator's manual for corresponding dialysis machine |
| pH (prepared solution) | Approx. 11 | Approx. 2 |
| Residual test needed | No | No |
| Biodegradable content | Yes | Yes |
| Storage conditions | Store below +30°C (+86°F) | Store below +30°C (+86°F) |
| Shelf life | 24 months from date of manufacture | 24 months from date of manufacture |
| Product code | 955850 | 955851 |
| Packaging | Box of 33 cartridges | Box of 33 cartridges |
| Box dimensions | 279 x 247 x 81 mm (11.0" x 9.7" x 3.2") | 279 x 247 x 81 mm (11.0" x 9.7" x 3.2") |
| Total box weight | 1200 grams (2.65 lbs.) | 1800 grams (3.97 lbs.) |

Rx only. For safe and proper use of these devices, refer to the Operator's Manual.

References:

1. Data on file. Baxter Healthcare Corporation.
2. AK 98 Operator's Manual P/N HC12839003.
3. Pancirová J. EDTNA ERCA J. 2001;27(3):123-124.
4. CleanCart A Operator's Manual P/N 0719001609.
5. CleanCart C Operator's Manual P/N 0719001608.